

Cleaning Procedure for the Eyegaze Edge®

The Eyegaze Edge must be disinfected prior to loaning it to a user. The following procedure will clean the system without harming it. *Please note: this procedure is intended only for cleaning the Eyegaze Edge. We cannot guarantee it will be safe to use on other systems.*

Step 1:

Disconnect the camera from the Eyegaze tablet.



Step 2:

Wipe the tablet surface with 70% or 90% isopropyl alcohol. Wipe the tablet's front, back, and edges, paying close attention to ensuring no alcohol gets into the ports. Putting alcohol inside of a port can cause serious damage to the computer.



Step 3:

Wipe all the camera surfaces with isopropyl alcohol, being sure to avoid getting alcohol into the port and camera aperture.



Step 4:

Wipe all cables and connectors with isopropyl alcohol.



Step 5:

Clean the mount, arm, and keyboard. You may use either isopropyl alcohol or any antibacterial wipe, being sure to wipe all surfaces.



Step 6:

Be sure all components are completely dry before packing.

